

DermaScar® ULTRA PLUS C&E Silicone gel for scar reduction with added Vitamin C&E

What is DermaScar® ULTRA PLUS C&E gel?

DermaScar® ULTRA PLUS C&E is an advanced, topical silicone gel that is transparent, dries quickly and helps maintain the skin's moisture balance, while aiding the management of scarred skin surfaces resulting from surgery, burns and other injuries.

Our premium silicone gel contains the antioxidants Vitamin C and E. DermaScar® ULTRA PLUS C&E lightens, hydrates and flattens scars. DermaScar® ULTRA PLUS C&E relieves the itching and discomfort of scars, as well as reducing the discolouration associated with scars. DermaScar® ULTRA PLUS C&E gel is easily applied to all areas of the skin, including the face, joints and flexures and dries rapidly to form an individual "sheet". DermaScar® ULTRA PLUS C&E was developed using the same basic long chain polymers as topical silicone gel sheeting, but is available in a tube. It contains polysiloxane, 1% Vitamin C ester and 10% Vitamin E. The gel is available in tubes of 15g.

What is DermaScar® ULTRA PLUS C&E gel used for?

DermaScar® ULTRA PLUS C&E is used to aid in the management of keloids and hypertrophic scars (such as those resulting from general surgical procedures, trauma wounds and burns), after the wound has healed and the skin surface is intact.

How should DermaScar® ULTRA PLUS C&E gel be used?

Wash the affected area and pat dry. DermaScar® ULTRA PLUS C&E should be applied to clean, dry skin only. Gently massage a small amount of DermaScar® ULTRA PLUS C&E into the scar. If you apply too much gently wipe away any excess with a tissue in order to avoid staining of clothing. Apply twice daily, in the morning and evening. Once dry, DermaScar® ULTRA PLUS C&E may be covered with cosmetic blush.

DermaScar® ULTRA PLUS C&E should not be:

- Applied to open or fresh wounds.
- Placed in contact with mucous membranes or applied too close to the eyes.
- Applied over antibiotic skin preparations or other skin treatments and/or products.

Are there any possible unwanted effects of DermaScar® ULTRA PLUS C&E gel?

Rarely, the gel may cause redness, pain or irritation to the skin. If you are concerned about these or any other unwanted effects, consult your pharmacist or doctor.

How should DermaScar® ULTRA PLUS C&E be stored?

- Keep out of reach of children.
- Keep gel away from heat.
- Store below 25°C.
- Do not use after the expiry date (EXP) printed on the pack.

Further Information

You can get more information about DermaScar® ULTRA PLUS C&E from your pharmacist or doctor. Information is also available on the product website; www.dermascar.com.au

Manufactured and Distributed by:

First Pharma Pty Ltd
Level 10, 56 Pitt Street
Sydney NSW 2000 Australia
Medical Enquiries: 1800 323 333

Made in Australia